

PRONTO

How to choose the right size

Your nose is as unique as your finger print so we have created a sizing guide to help you select the right Pronto for you.

Simply print out this page and carefully cut around the dotted line of a sizing guide and follow the steps below to size your nose.

When printing, select **'Actual Size'** under page sizing options to ensure guides are printed to scale.

TIP: Generally, medium is the most common size so we suggest you cut out the medium guide first.



1 If you're unsure which size to start with, cut out the medium sizing guide.

Then with the colored side facing away from you, position the sizing tool under your nose.

2 For a good fit, the black tip should be located at the crease line. In this example, **Pronto Medium** would be the right size.

If the black tip sits above the crease line, then **Pronto Small** would be the right size for you.

3 When Pronto is the right size and is worn correctly, only the scent diffuser should be showing outside the nose.

Please note that this is a guide only. The external size of your nose is not always representative of the internal anatomy. A pronounced deviation or internal scar tissue caused by an injury may mean that you cannot fully insert the size indicated by the guide. You may need to choose one size smaller than indicated.

AFTER PRINTING, THE SCALE IS CORRECT IF THIS LINE



MEASURES 37MM (1.46")

